

LARGE DIAMETER LOW PROFILE



Protégé™ GPS™
Self-expanding Peripheral
Stent System

Compact delivery

- Diameters up to 14 mm
- Full line is 6 F compatible



Precision

- Proprietary EX.P.R.T.™ deployment system secures the stent to eliminate premature deployment or “jumping”
- Tantalum GPS markers enhance visibility for easier, precise positioning

Radial strength and flexibility

Designed for radial strength without sacrificing flexibility.



DURABILITY ILIAC Clinical Trial Summary (nine-month data)

STUDY OBJECTIVE

To confirm the safety and effectiveness of primary stenting using the EverFlex™ and Protégé™ GPS™ self-expanding stent systems for the treatment of lesions in the common and external iliac arteries.

STUDY DESIGN

- Prospective, multicenter, nonrandomized
- 75 subjects enrolled
- Primary outcome: major adverse event (MAE) rate at nine months
- Clinical follow-up at predischarge; 30 days; nine months; 1, 2 and 3 years post procedure
- Independent Clinical Events Committee (CEC) and core laboratory analysis

PRIMARY OUTCOME (N = 75)

EverFlex™ and Protege™ GPS™ Stents

The MAE rate at nine months defined as a composite of periprocedural death, in-hospital myocardial infarction (MI), clinically driven target lesion revascularization (TLR), and amputation of the treated limb through nine months post procedure.

1.3%

Nine-month MAE

0.0%

Periprocedure death

0.0%

In-hospital MI

1.3%

Clinically driven TLR

0.0%

Amputation of the treated limb

CONCLUSION

Nine-month results of the DURABILITY Iliac study showed favorable safety and efficacy, with a MAE rate of 1.3% and **A PRIMARY PATENCY RATE OF 95.8%.**

Protégé™ GPS™

Self-expanding Peripheral Stent System

USABLE SHAFT LENGTH		STENT SIZE		COMPATIBILITY		
Product catalog number 80 cm	Product catalog number 120 cm	Diameter (mm)	Length (mm)	Recommended introducer sheath (F)	Guidewire (in)	Recommended lumen size (mm)
SERP65-09-20-80	SERP65-09-20-120	9	20	6	0.035	7.5 - 8.5
SERP65-09-30-80	SERP65-09-30-120	9	30	6	0.035	7.5 - 8.5
SERP65-09-40-80	SERP65-09-40-120	9	40	6	0.035	7.5 - 8.5
SERP65-09-60-80	SERP65-09-60-120	9	60	6	0.035	7.5 - 8.5
SERP65-09-80-80	SERP65-09-80-120	9	80	6	0.035	7.5 - 8.5
SERP65-10-20-80	SERP65-10-20-120	10	20	6	0.035	8.5 - 9.5
SERP65-10-30-80	SERP65-10-30-120	10	30	6	0.035	8.5 - 9.5
SERP65-10-40-80	SERP65-10-40-120	10	40	6	0.035	8.5 - 9.5
SERP65-10-60-80	SERP65-10-60-120	10	60	6	0.035	8.5 - 9.5
SERP65-10-80-80	SERP65-10-80-120	10	80	6	0.035	8.5 - 9.5
SERP65-12-20-80	SERP65-12-20-120	12	20	6	0.035	9.5 - 11.0
SERP65-12-30-80	SERP65-12-30-120	12	30	6	0.035	9.5 - 11.0
SERP65-12-40-80	SERP65-12-40-120	12	40	6	0.035	9.5 - 11.0
SERP65-12-60-80	SERP65-12-60-120	12	60	6	0.035	9.5 - 11.0
SERP65-12-80-80	SERP65-12-80-120	12	80	6	0.035	9.5 - 11.0
SERP65-14-20-80	SERP65-14-20-120	14	20	6	0.035	11.5 - 13.0
SERP65-14-30-80	SERP65-14-30-120	14	30	6	0.035	11.5 - 13.0
SERP65-14-40-80	SERP65-14-40-120	14	40	6	0.035	11.5 - 13.0
SERP65-14-60-80	SERP65-14-60-120	14	60	6	0.035	11.5 - 13.0
SERP65-14-80-80	SERP65-14-80-120	14	80	6	0.035	11.5 - 13.0

Medtronic

Invatec S.p.A.
Via Martiri della Libertà 7
25030 Roncadelle (BS)
Italy
Tel: +39.030.2589311

Medtronic International Ltd.
49 Changi South Avenue 2
Singapore 486056
Tel: +65.6436.5000

Medtronic
710 Medtronic Parkway NE
Minneapolis, MN 55432
USA
Tel: +1.763.514.4000

Medtronic Australasia Pty Ltd.
97 Waterloo Road
North Ryde, NSW 2113
Australia
Tel: +61.29857.9000

Medtronic International Trading Sàrl
Route du Molliou 31
CH-1131 Tolochenaz
Switzerland
Tel: +41.21.802.7000

Medtronic New Zealand
Unit N16, Mezzanine Level 5
Gloucester Park Road
Onehunga
Auckland
New Zealand

Medtronic of Canada Ltd.
99 Hereford Street
Brampton, Ontario L6Y 0R3
Canada
Tel: +1.905.460.3800

Medtronic Korea Co., Ltd.
5F, Sajo Building
1001 Daechi-dong, Kangnam-ku
Seoul, 135-280
Korea
Tel: +82.2.3404.3600

Medtronic Latin America
3750 NW 87th Avenue, Suite 700
Miami, FL 33178
USA
Tel: +1.786.709.4200

Medtronic META FZ-LLC
Office Park, Block D, 2nd Floor
PO Box 500638
Dubai Internet City | Dubai,
United Arab Emirates
Tel: +971.4.818.2666

medtronic.com/apv

DC00042011 © 2016 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. Not for distribution in the USA, Japan or France. 05/16